

# MinXray P200D MKIII

## TECHNIQUE CHART Exposure time and angulation of central ray

		Exposure Time (sec)		
		Adult	Child	Angle
<b>Maxillary</b>	<b>Molars</b>	0.8	0.5	+20°
	<b>Bicuspid</b>	0.8		+30°
	<b>Cuspids</b>	0.6	0.4	+45°
	<b>Incisors</b>	0.6	0.3	+40°
<b>Mandibular</b>	<b>Molars</b>	0.6	0.4	-5°
	<b>Bicuspid</b>	0.6		-10°
	<b>Cuspids</b>	0.4	0.3	-20°
	<b>Incisors</b>	0.4	0.3	-15°
<b>Maxillary</b>	<b>Incisor Region</b>	1.0		+65°
<b>Occlusals</b>	<b>Cuspid-Molar Region</b>	1.0		+60°
<b>Mandibular</b>	<b>Incisor Region</b>	1.0		-55°
<b>Occlusals</b>	<b>Cuspid-Molar Region</b>	1.0		-90°

*Notes:*

*KODAK Ultra-Speed film*

*Exposure times are intended as guides only. They are based on averages and may be modified to suit the individual patient. The decrease should be made in steps of 20 - 25%.*