

# HF80H

# TECHNIQUE CHART\*

X-ray Unit: MinXray, Inc., **Model HF80H**

Screen/Film Combination: 800 Speed

Focal Film Distance: 40 Inches (100 cm.)

Patient: 190 lbs. (86.3 kgs.)

EXAM	kV	TIME	GRID
AP Skull	80	.30	6:1, 85L
Lat Skull	70	.26	6:1, 85L
AP Chest	80	.10	None
Lat Chest	80	.22	6:1, 85L
Shoulder	65	.12	None
Elbow	60	.08	None
AP Hand	55	.08	None**
Lat Hand	55	.08	None
C-Spine	70	.30	6:1, 85L
AP T-Spine	80	.30	6:1, 85L
Lat T-Spine	80	.60	6:1, 85L
AP L-Spine	80	.60	6:1, 85L
Lat L-Spine	80	1.20	6:1, 85L
AP Abdomen	80	.44	6:1, 85L
AP Pelvis	80	.40	6:1, 85L
Lat Hip	80	.44	6:1, 85L
Femur	75	.30	6:1, 85L
Knee	70	.10	None
Ankle	60	.10	None
AP Foot	60	.08	None
Lat Foot	65	.10	None

*\*These are recommended starting techniques.  
Final results depend upon many factors.  
Therefore, if films are too dark (overexposed),  
decrease time; if films are too light  
(underexposed), increase time.*

*\*\*FFD: 46" (116.8 cm)*

# MINXRAY<sup>INC.</sup>

3611 Commercial Avenue,  
Northbrook, IL 60062-1822, USA  
Telephone 847-564-0323  
Toll Free 1-800-221-2245  
FAX 847-564-9040